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USER INSTRUCTIONS FOR THE DYNAMIC POSITIONING ORTHOSIS BY REBACARE®

Support and comfort with the cushion range (positioning cushions) by REBACARE®

- Before using the positioning cushions it is important to clean the outside with 1000 mmp chlorine, 70% alcohol. See also the guidelines of the Infection Prevention Working Group (WIP) (www.rivm.nl)
- The positioning cushions are a versatile product.
- Ask your occupational therapist or physiotherapist how to correctly position the cushions.
- Before positioning it is important to evenly divide the stuffing. Positioning the cushion correctly may prevent pressure sores by dividing the pressure or releasing parts.
- The positioning cushions help you relax because they support you in a comfortable position and relieve pressure.
- The positioning cushions can not be washed in a washing machine or tunnel washer.
- The cover (case) can be washed at 60 degrees. The cover (case) of the hospital type can be washed at 90 degrees. Both covers can be dried in a tumble dryer.
- The positioning cushions are not suitable for people with epilepsy due to PU ticking.
- Watch our useful instruction video: www.rebacare.com

Please ensure that the patient indicates that the relevant cushion feels comfortable!

REBACARE GIVES A 2-YEAR PRODUCT WARRANTY

REBACARE recommends very intensive use after a period of 2 years regarding cleaning and hygiene (incontinence) to the product regularly to inspect and replace if necessary.