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Foto 3

## USER GUIDE FOR THE NUN BY REBACARE®

- Before using the models, it is important to clean the outside with 1000 ppm chlorine, 70% alcohol, OxyFoam, OxyWipe or detergent. Or wash at 95 degrees. See also the guidelines of the Infection Prevention Working Group (WIP) ([www.rivm.nl](http://www.rivm.nl))
- Nun is not suitable for children aged 3 and under.
- Nun does not contain any latex and can be disposed of with regular household waste.
- Nun is placed deep in the Orthopedic Bandage shoe /heel protector from REBACARE® after the bandage has been applied.
- Nun does not touch the skin.
- Nun cannot be physically attached to the shoe, but it does stay in place thanks to the perfect form.
- Intended use: Nun is a useful tool that can be used to protect the Orthopedic Bandage shoe / heel protector by REBACARE® in the case of exuding wounds.

Do not dry the Nun by REBACARE® in a tumble dryer. Clean before use.

- The patient's name can be written on the care label using a ballpoint pen.
- Can be used for both the right foot and the left foot.
- Bandage the foot with the exuding wound (Image 1).
- Place the patient's leg in the Nun and then place both in the REBACARE® Orthopedic Bandage shoe /heel protector (Image 2).
- The post-operative shoe's grey sole should be turned to the outside, so it can perform its anti-slip function. The grey sole is on the bottom of the Orthopedic Bandage shoe / heel protector.
- Position the lower straps across the instep of the foot and bring the left-hand strap over to the right. Then place the right-hand strap over the left-hand strap.
- Position the upper straps across the front of the shin and bring the right-hand strap over to the left. Then place the left-hand strap over the right-hand strap. The end result should be as shown in Image 3.
- Please also watch our useful instruction video: [www.rebacare.com](http://www.rebacare.com)

Nun

REBACARE®  
Sasker van Heringawei 12  
9034 GD Marsum

The Netherlands  
[info@rebacare.com](mailto:info@rebacare.com)  
[www.rebacare.com](http://www.rebacare.com)

## REBACARE® GIVES A 6 MONTHS PRODUCT GUARANTEE

After a 6 months period of intensive use in terms of cleaning and hygiene (exudate), REBACARE® recommends that the product is regularly inspected and replaced if necessary.

### ⚠ WARNING:

- **Only use the Nun in the Orthopedic Bandage shoe /heel protector from REBACARE® and do not combine it with other products.**
- **Any wound treatment of the patient should be performed before using the Nun.**
- **Nun is placed deep in the Orthopedic Bandage shoe /heel protector from REBACARE® after the bandage has been applied. Nun does not touch the skin.**
- **Check with the patient that the Nun and the Orthopedic Bandage shoe / heel protector feel comfortable to them!**
- **Any serious incident related to the patient device must be reported to REBACARE® and the competent authority of the member state where the patient resides.**

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