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Version 1.1

USER INSTRUCTIONS FOR THE DYNAMIC POSITIONING ORTHOSIS FREE EAR® EAR CUSHION BY REBACARE®

The innovative Free Ear® ear cushion by REBACARE® is patented and leaves the ear free. The Free Ear® ear cushion is used for pressure sores of the ear.

- Before use, clean the cushion completely on the outside with 1000 mmp chlorine, 70% alcohol. See also the guidelines of the Infection Prevention Working Group (WIP) (www.rivm.nl)
- The Free Ear® ear cushion is a versatile product.
- Place the patient's head on the Free Ear® ear cushion so that the ear is positioned in the cut-out area.
- In the image of the Free Ear® ear cushion shown here (see image) the cushion is suitable for the left ear.
- To use the Free Ear® ear cushion of the patient for the right ear, the Free Ear® ear cushion has to be turned from right to left.
- Ensure that the face of the patient is directed toward the cut-out part.
- The patient's neck is on the narrowest part of the Free Ear® ear cushion.
- The Free Ear® ear cushion is not secured, but is loose on the bed.
- REBACARE® recommends changing the position of the patient every 3 hours, and if there are no medical objections, to turn the patient over so that the other ear can be positioned on the Free Ear® ear cushion.
- The Free Ear® ear cushion is not suitable for people with Epilepsy, due to PU ticking.
- The Free Ear® ear cushion can not be washed in a washing machine or tunnel washer.
- The cover (case) can be washed at 60 degrees. The cover (case) of the hospital type can be washed at 90 degrees. Both covers can be dried in a tumble dryer.
- Please also watch our useful instruction video: www.rebacare.com

Ensure the patient indicates that the Free Ear® ear cushion feels comfortable!

Version 1.1