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USER MANUAL REBACARE® DYNAMIC POSITIONING ORTHOSIS (DPO)

- Before using the models, it is important to clean the outside with 1000 ppm chlorine, 70% alcohol, OxyFoam, OxyWipe or detergent. See also the guidelines of the Infection Prevention Working Group (WIP) (www.rivm.nl)
- Ask your occupational therapist or physiotherapist how to correctly position the models.
- Before positioning the models, it is important to evenly divide the stuffing. By correctly positioning and passing a hand between the patient and the model, excess air can be drawn out through the filters. The filters will close automatically. This creates an individual mould of the body. The patient is supported comfortably and completely, without overpressure.
- The models cannot be washed in a washing machine or tunnel washer.
- For an explanation of the different applications, please refer to our website: www.rebacare.com/instructions
- The models are not suitable for children aged 3 and under.
- Our products do not contain any latex and can be disposed of with regular household waste.
- Please ensure that the patient indicates that the relevant cushion feels comfortable!
- Intended use: positioning users/patients (over 3 years old) to prevent and/or treat decubitus in the following indication(s): paraplegia, CVA, ALS, MS, Parkinson's disease, congenital abnormalities, comfort, dementia and/or conditions where postural management is desired, for example in Intensive Care Units (ICU), nursing homes, hospice or at home.

Luna

Twister

Forma

Crescent

Maxima

Support

Comfort

Comfort XL

Duorole

Minima

Role

Free Ear®
Ear pillow

Multi 54

Multi 64

MEDICAL PRODUCT CAN BE REUSED
AFTER CLEANING

CHECK THE PRODUCT FOR ANY DAMAGE

DISPOSE OF THE PRODUCT IF DAMAGED

DISINFECT WITH 70% ALCOHOL AND MAX.
1000 PPM CHLORINE, OXYFOAM,
DETERGENT OR OXYWIPE

WIPE THE PRODUCT WITH A CLOTH
MOISTENED WITH WATER

REBACARE® GIVES A 2-YEAR PRODUCT GUARANTEE

After a two-year period of intensive use in terms of cleaning and hygiene (incontinence), REBACARE® recommends that the product is regularly inspected and replaced if necessary.

REBACARE® gives a 6-month guarantee on the covers. These covers must be washed before use in accordance with the washing instructions. REBACARE® has 3 different matching covers:

Standard Nicky stretch covers (80% cotton and 20% polyester, can be washed at 60 degrees Celsius)

Stretch covers (100% cotton, can be washed at 60 degrees Celsius)

Shrink-free covers without stretch (100% cotton, can be washed at 95 degrees Celsius)

WARNING:

- Oral or nasal ingestion of the model contents (polystyrene beads and/or siliconised hollow fibre) may cause respiratory problems. When repositioning the patient, ensure they are lying in a 30-degree lateral position (if possible). This prevents the patient turning too far and unintentionally ending up in a prone position.
- Due to potential leakage of the contents, REBACARE® models may only be used with the original REBACARE® covers. These original covers are not included and can be ordered separately.
- The head models contain polyurethane ticking. Therefore they are not suitable for people with epilepsy.
- Any serious incident related to the patient device must be reported to REBACARE® and the competent authority of the member state where the patient resides.
- Do not combine parts with other positioning systems.
- Any wound treatment of the patient should be performed before using the models.

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