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Foto 1

Foto 2

**MANUAL OF THE BANDAGE SHOE / HEEL PROTECTOR  
BY REBACARE®**

✗ The bandage shoe / heel protector can not be used in the drying machine. This will make the anti-slip function of the sole disappear, and will harden the sole as well.

- 60 • The bandage shoe / heel protector is a personal product. Washing the shoe at 60°C will ensure a long lifespan.
- 80 • The bandage shoe / heel protector can be washed at 80°C for about ten minutes. This will influence the lifespan from about ten washes.
- The name of the patient can be written on the label using a ballpoint.
- The inlay sole (can be delivered upon request) can be taken out of the shoe and can be adjusted to the right size with a pair of scissors.
- Before washing the bandage shoe / heel protector, the Velcro should be closed.
- The leg should be placed in the bandage shoe / heel protector as shown in picture 1.
- Make sure that the grey sole of the bandage shoe / heel protector is placed on the outside of the shoe to ensure the anti-slip function.
- The lower bands should be placed on the instep of the foot. The right band should be brought to the left. Subsequently, the left band should be put over the right band, as exhibited on picture 2.
- Place the upper bands on the front of the shin. The left band should be brought to the right. Then, put the right band over the left band, as pictured in the 2nd photograph.
- Watch our useful instruction video: [www.rebacare.com](http://www.rebacare.com).

**Make sure that the patient gives an indication of whether the bandage shoe / heel protector is comfortable!**

Picture 1

Picture 2