


SUPINE METHOD - INTENSIVE CARE (IC) / MEDIUM CARE (MC)




- IC / MC
- Distress at night due to repositioning
- Spine, tailbone and heel decubitus

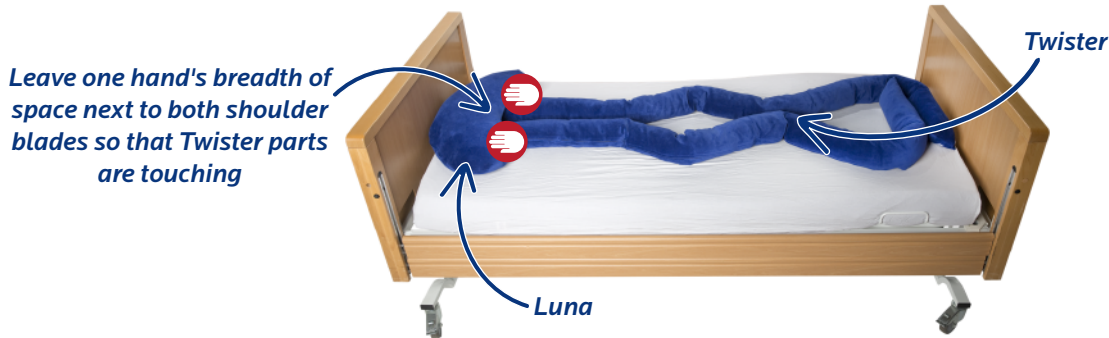

Optionally, apply repositioning by removing one role for a 12-degree lateral position. Put Minima under the knee that is touching the mattress

Fold line

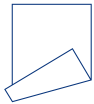

- 1**
No force (do not forcefully push or pull on the patient).
- 2**
Always leave a hand's breadth of space.
- 3**
Fill all of the spaces where the patient's body does not make contact with the pillow.
- 4**
Smooth the area where the patient touches the pillow to relieve any air pressure.
- 5**
Provide opposite support to the patient in a lateral position.